

# Safely Spike Cube



Perfect for patients with weak hands or grip but designed for everyone.

- Helps patients safely spike saline solution bag.
- Reduces the chance of stabbing the saline bag.
- Helps prevent accidental contamination or stabbing through the saline bag causing waste of supplies and contamination of the spike.

## Cube Instructions



To Install: Place the Safely Spike Cube on to the saline bag with the wings falling into the slot. Remove the tab.



To spike the bag: Place the spike into the port and press the spike into place using the Safely Spike Cube to support the port.



Ease of use: The saline bag has been safely spiked and the Safely Spike Cube can now be removed.